







## PURELL® Antimicrobial Wipes Plus

**Antiseptic wipes for surfaces**

**Target markets:** Hospital, health centre, laboratories, emergency services, leisure facilities

- Active against Coronavirus according to EN 14476
- Broad spectrum activity (bacteria, yeast, selected viruses)
- Easy-to-use

### TECHNICAL INFORMATION

<b>Active substance</b>	Ethanol CAS 64-17-5 (48 g/kg), Benzalkonium chloride CAS 85409-22-9 (3 g/kg), Chlorhexidine digluconate CAS 18472-51-0 (1,2 g/kg)			
<b>Specifications</b>	<ul style="list-style-type: none"> <li>• Non woven: white spunbonded, 22g/m<sup>2</sup></li> <li>• Fragrance: mentholated</li> </ul>			
<b>Preservation</b>	<ul style="list-style-type: none"> <li>• Before opening, wipes may be stored for 24 months away from heat, under normal conditions of temperature</li> <li>• Store at room temperature in its original, closed packaging</li> <li>• Avoid temperatures above 40 °C</li> </ul>			
<b>Directions for use</b>	<p>1. To pull the first wipe, remove the canister lid and pull the first wipe from the centre of the reel with clean hands. Replace the canister lid with the first wipe pulled through the opening in the lid.</p> 	<p>2. Wipe the surfaces to be disinfected until visibly wet. Use several wipes if necessary. Do not use wipe if dry. Close the lid after use.</p> 	<p>3. Leave to act until required level of antimicrobial efficacy is achieved. Do not rinse.</p> 	<p>4. Dispose of wipes in appropriate waste container</p> 
<b>Regulation</b>	<ul style="list-style-type: none"> <li>• Biocidal product PT2</li> <li>• For professional use only</li> </ul>			
<b>Environment</b>	<ul style="list-style-type: none"> <li>• Recyclable packaging</li> <li>• Dispose of used wipes in a waste container</li> <li>• Do not dispose of the product into the environment</li> </ul>			
<b>Precautions for use</b>	<ul style="list-style-type: none"> <li>• Do not swallow</li> <li>• Do not mix with other products</li> <li>• For a use on smooth surfaces, objects and small materials</li> <li>• The use of wipes does not replace following the cleaning protocol (prerequisite to any disinfection)</li> <li>• Do not reuse a wipe as this could cause cross contamination</li> <li>• Do not use a wipe if dry</li> <li>• For further information, refer to the product safety data sheet</li> <li>• In case of emergency, contact the nearest poison control center</li> </ul>			

## MICROBIOLOGICAL PROPERTIES OF THE SOLUTION


Active against	Standard	Contact time
Bacteria	EN 1276* (v.2010)	30s
	EN 13727* (v.2015)	30s
	EN 13727* (v.2015), <i>Staphylococcus aureus</i> (MRSA), Vancomycin-resistant <i>Enterococcus faecium</i> (VRE), Extended-spectrum <i>Escherichia coli</i> (ESBL)	30s
	EN 13697** (v.2015)	1 min
	EN 16615* (v. 2015), <i>P.aeruginosa</i> , <i>S. aureus</i> , <i>E. hirae</i>	15 min
Yeast	EN 13624* (v.2013), <i>Candida albicans</i> + <i>Candida auris</i>	1 min
	EN 13697* (v.2015), <i>Candida albicans</i>	5 min
	EN 13697* (v.2019), <i>Candida auris</i>	5 min
	EN 1650* (v.2013), <i>Candida albicans</i>	1 min
	EN 16615* (v. 2015), <i>Candida albicans</i>	15 min
Virus	Active against:	
	EN 14476* (v.2019), Coronavirus BCoV (surrogate virus)	30s
	EN 14476* (v.2015), Rotavirus	1 min
	EN 14476* (v.2015), Vaccinia virus (MVA) Also covering the following enveloped viruses: Hepatitis B, Hepatitis C, Influenza virus, HIV...	30s

\* Study conducted in clean conditions.

\*\* Study conducted in dirty conditions.

PACKAGING	WIPE FORMAT	PRODUCT CODE
PURELL Antimicrobial Wipes Plus – 100 Count Canister x 12	130 x 210 mm	9202-12-EEU00
PURELL Antimicrobial Wipes Plus – 270 Count Canister x 6	150 x 170 mm	9213-06-EEU00
PURELL Antimicrobial Wipes Plus – 1200 Count Refill x 2	150 x 200 mm	9218-02-EEU00

Use biocides safely. Always read the label and product information before use.

 <p>GOJO Industries - Europe, Ltd.* Units 5 &amp; 6 Stratus Park Brinklow Milton Keynes MK10 0DE UK Tel: +44 (0) 1908 588444 Fax: +44 (0) 1908 588445</p>	<p>Laboratoires Prodene Klint – a GOJO Family Company 8 Rue Léon Jouhaux 77183 Croissy-Beaubourg – France Tél.: +33 (0)1 60 95 49 00 Fax.: +33 (0)1 60 95 49 49</p>	<p>GOJO Iberia España, S.L. Paseo de la Castellana, 90, 1ª planta, 28046 Madrid, Espagne Tel: +351 218 988 230 Fax: +351 21 898 82 39</p>	<p>GOJO Switzerland Sàrl Chemin du Faubourg-de-Cruseilles 11, c/o RSM (Gva) SA, 1227 Carouge GE +41 225706057</p>	<p>Prodene GmbH – a GOJO Family Company Im Trutz Frankfurt 55 60322 Frankfurt, Allemagne Tel.: +49 (0) 6074 9196057 Fax: +49 (0) 6074 9196685</p>
-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------------------